135mm

210mm

VELVET is a women-oriented barrier contraception.  If used correctly and retained in vagina during sexual intercourse, it provides dual protection against pregnancy and sexually transmitted infections.

To ensure optimal use, see warnings and carefully follow instructions for use in the leaflet inside.

Statement of Limitation: "No barrier methods, including male and female condoms provide 100% protection against pregnancy and STIs."

 2460

 Each condom for single use only.

Mfg. Lic. No. 03/25/2005 

ISO 25841: 2014

REV NO : 01/FCF01/05/2017

 Superior Lubricated Latex Condoms  
Woman Initiated Contraception

**VELVET**  
CONDOMS FOR WOMEN

**CONDOMIZE!**

 Open the packet

 Outer ring covers opening of the vagina

 Squeeze the sides

 Push the inner ring into the vagina

 Use lubricant to push condom inside

 Guide your partner's penis into VELVET

 After sex ends and pull it out

Disposal: Wrap the condom in the sachet and throw it in dustbin. Do not flush it.

 Manufactured by HLL Lifecare Limited  
Plot no. 16/A-1, Cochin Special Economic Zone,  
Kakkanad, Cochin 682 037, Kerala, India.  
e-mail: [info@hlllifecare.com](mailto:info@hlllifecare.com)  
Customer Care Number: +91 484 2413999

 OBELUS S.A.  
Rue General Wabbe, 53  
1030 Brussels, Belgium  
Tel: +32 2 732 59 51 Fax: +32 2 732 60 03  
Email: [mail@obelus.net](mailto:mail@obelus.net) Web: [www.obelus.net](http://www.obelus.net)

25 MM

36.5 MM

6 MM

75 mm

6 MM

36.5 MM

25 MM

10mm

5mm

**CUPID LIMITED**  
Manufacturer and Suppliers of Male, Female Condoms & Lubricants

CONDOMIZE\_CUPID FC

ART WORK NAME	CONDOMIZE_CUPID FEMALE CONDOM _POUCH ART WORK
ART WORK NO	AW-FC-PP-13-Rev.00 DATED: 12/10/2017
SIZE	100X127 MM_
DATE	12/10/2017
COLORS	

FRONT SIDE

100.MM

88.MM

6mm  
sealing  
area

6mm  
sealing  
area

Notch Here 25 mm

127.MM

1 Lubricated Latex Female condom

CONDOMIZE!

**CUPID**  
FEMALE CONDOM

**CONDOMIZE!**

**UNFPP**

**DIRECTIONS FOR USE:**

Step 1: Open the female condom package carefully, tear at the notch on the top right of the pack-age. Do not use scissor or a knife to open.

Step 2: While holding the sheath at the closed end, grasp the sponge and squeeze it with the thumb and second finger. Push the sheath into the vagina. The female condom should be inserted into the vagina before any contact occurs between the vagina and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy. Do not use Female Condoms with oil based lubricants. If necessary use water based lubricants. Trim the nail of the finger used for insertion of female condom to avoid tear of condom while insertion. Do not use Male and Female Condoms together.

Step 3: Before use, ensure that your partner's penis is not entering the vagina. The female condom is slipping into or out of the vagina, immediately stop and check.

Step 4: Place the index finger on the inside of the condom to assist in the prevention of STIs and pregnancy.

Step 5: The female condom is now in place and ready for use with your partner.

Step 6: Be aware that your partner's penis is not entering the vagina. The female condom is slipping into or out of the vagina, immediately stop and check.

Step 7: Withdraw the penis soon after ejaculation leaving the female condom in place in the vagina. To remove the female condom, after the use, twist the outer ring and gently pull the condom out.

Step 8: Wrap the condom in the pack age or in tissue and throw it in the garbage. Do not put it into toilet.

For more information on condoms or STIs, contact a health care provider or public health agency. If the female condom is leaked or burst during use, seek medical assistance as soon as possible, at least within 72 hours. If this container is damaged, discard the female condom & use a new one from a undamaged package. This product is manufactured in compliance with all requirements of ISO 25841:2014.

Neutral Code: MH/DRUG/NKD-48.  
Consumer Call Contact  
Tel No.: +91 2551 230280

AW-FC-PP-13-Rev.00 DATED: 12/10/2017

**CE** 2460

MADE IN INDIA

BATCH NO.: MFG. DATE: EXP. DATE:

ISO 9001:2015 ISO 13485:2015 ISO 14971:2007

ADHERENT LTD.  
HIGHER BUSINESS CENTRE-472  
TIC-04/4 207720050

ISO 9001:2015 ISO 13485:2015 ISO 14971:2007

EC REP

ISO 9001:2015 ISO 13485:2015 ISO 14971:2007

ISO 9001:2015 ISO 13485:2015 ISO 14971:2007

127.MM

Notch Here 25 mm

6mm  
sealing  
area

6mm  
sealing  
area

88.MM

100.MM

BACK SIDE

1  
2  
3  
4  
5  
6

The Female Health Company [www.fc2.us.com](http://www.fc2.us.com)

# CONDOMIZE!

## FC2 - Nitrile Female Condom with Ring

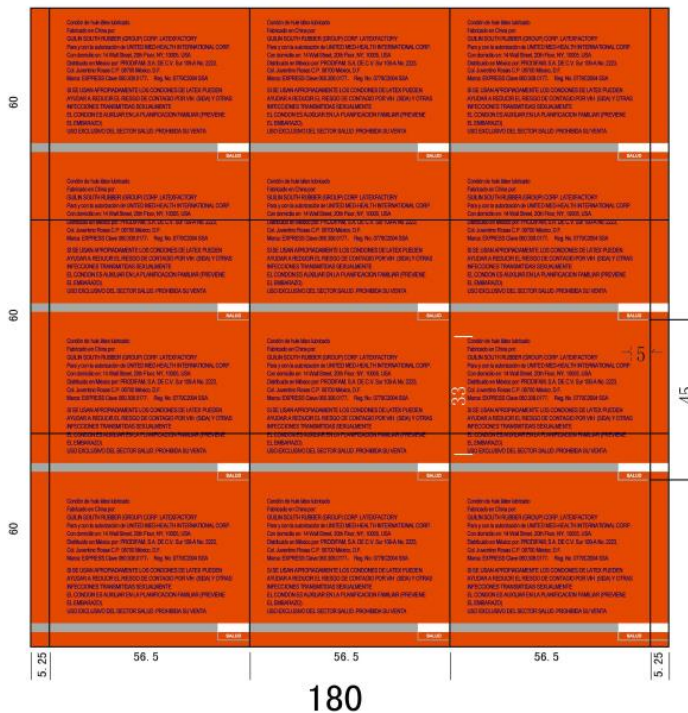
UNFPA


Because Everyone Counts

## Section IX: Condom Artwork Examples

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
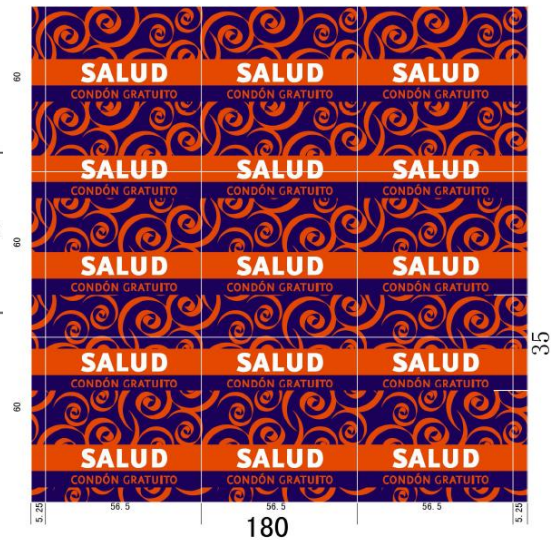
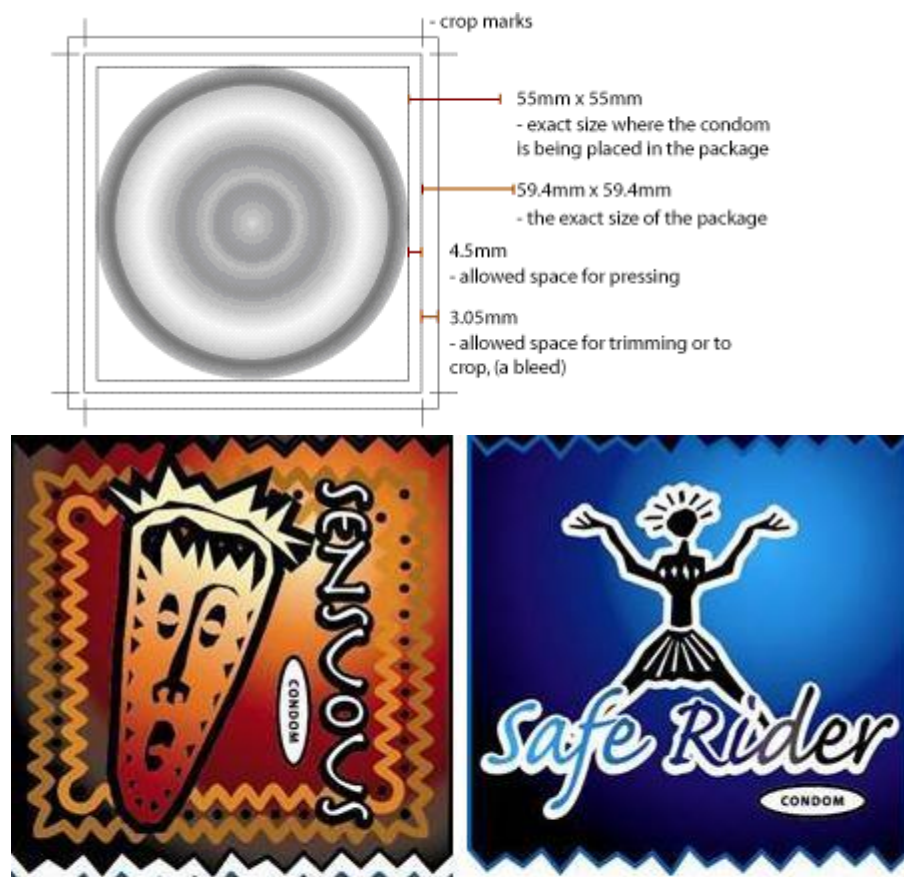
# UNFPA Condom Artwork – Artwork Property of UNFPA/Partner – do not duplicate


	C	M	Y	K		C	M	Y	K
	100	100	0	100		0	0	0	100
	91.37	95.69	18.82	6.67		38.04	94.12	90.98	38.43
	97.65	85.49	0	0		22.35	98.04	92.16	8.63
	91.37	27.06	0	0		2.75	65.1	91.37	0
	87.06	0	0	0		3.53	3.53	36.86	0

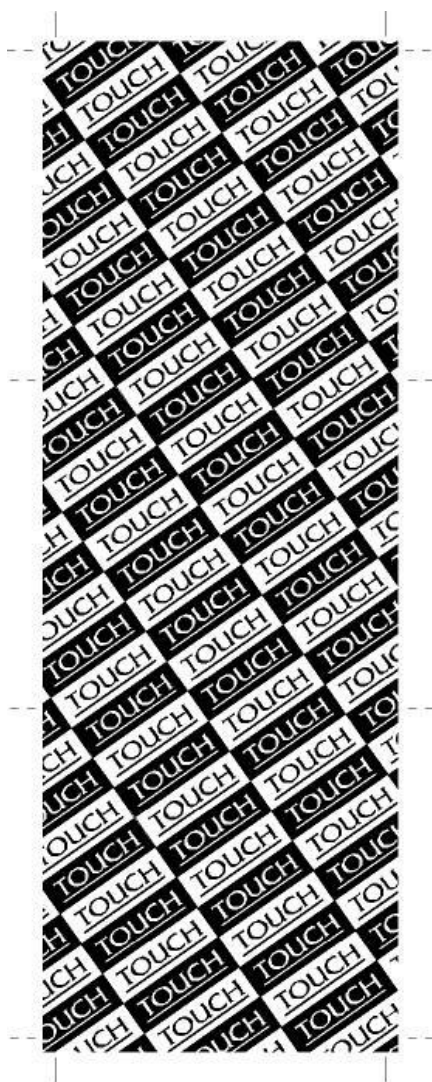PANTONE 2768 C

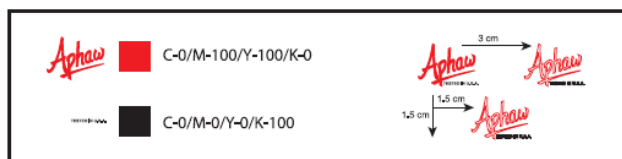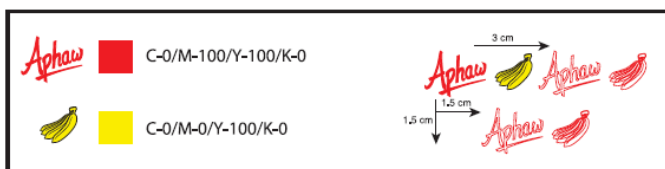
# UNFPA Condom Artwork – Artwork Property of UNFPA/Partner – do not duplicate

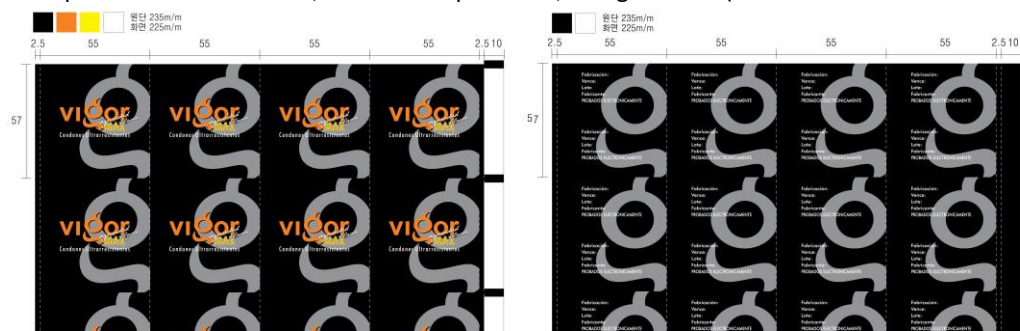Example of Artwork on a foil, individual square box, and gross box (2 condoms + 1 lubricant in each square box)

## UNFPA Condom Artwork – Artwork Property of UNFPA/Partner – do not duplicate

Lubricant in 56x56 mm (back & front)

Individual box – INSIDE & OUTSIDE

UNFPA Condom Artwork – Artwork Property of UNFPA/Partner – do not duplicate

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Distribuido en México por: PRODIGAM, S.A. DE C.V. Sur 109-A No. 2223,